## UreSil® THORA-VENT®

EN

### Occlusion Plug

#### REF: TVOCP

For use with Thoracic Vent

This package contains instructions for the specific use of the occlusion plug. For additional instructions related to the use of the Thoracic Vent, see the instructions for use that are included in the Thoracic Vent Procedure Trays.

#### INSTRUCTIONS FOR USE:

Check for continuous air leak: The Thoracic Vent occlusion plug can be used to check for a continuous air leak from the patient's lung. To occlude the Thoracic Vent, insert and secure the occlusion plug to the luer-lock on the self-sealing port.

Warning: Remove the occlusion plug when the check is complete. The Thoracic Vent will not vent air when the occlusion plug is in place. Seal the self-sealing port with the tethered cap after the occlusion plug is removed.

#### WARNINGS/COMPLICATIONS:

If the occlusion plug was used to perform the air leak check, remove the occlusion plug when the check is complete. The Thoracic Vent will not vent air when the occlusion plug is in place.

Sterile if package is unopened and undamaged.

Non-pyrogenic. For single use only.

Caution: Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician. TO REORDER:

Call 1-800-538-7374 (U.S.A. only) Fax: (847) 982-0106

UreSil and THORA-VENT are registered trademarks

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