
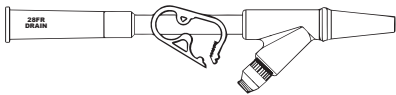
UreSil®
Aspiration/Flush Adapter
Large Bore Systems

en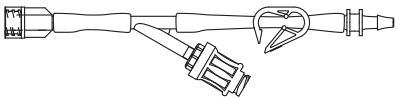

INSTRUCTIONS FOR USE:

1. For catheter maintenance, (aspiration and flushing), attach the Aspiration/Flush (A/F) Adapter to the Large Bore catheter hub (ex. UreSil's ULTI-flo LBL2 locking or LBNL non-locking).
2. Remove the connector from the selected drainage bag. Reconnect the bag tube to the barbed or funneled connector of the A/F Assembly (next to clamp).
3. Swab the silicone seal of the needleless port in accordance with facility protocol. Attach a male slip luer or male luer lock syringe to the needleless port by grasping the valve and firmly pushing and twisting the syringe onto the valve until tight.

CAUTION: Do not use needle to access the valve port.

4. To flush or aspirate the catheter, clamp the tube with the supplied clamp and complete the aspiration/flush as required. Unclamp the tube when complete and remove the syringe as required by twisting counter clock-wise until loose. Do not cap the valve. The device is closed when not engaged. For subsequent connections repeat from step three (3).

STERILE EO 0086

Sterile if package is unopened and undamaged. Non-pyrogenic. For single use only. The reuse of this single-use device can lead to patient infection and/or device malfunction. Caution: Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician. UreSil is a registered trademark of UreSil, LLC. U.S. Patent nos. 8,177,773, DES. 585,134S

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