
UreSil® ULTI-flo® en

General-Purpose Non-Locking Catheter

Catheter	Intended Use
General-purpose (GP™ Non-Locking)	Abscess and other fluid drainage.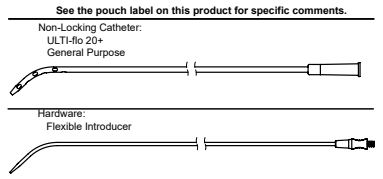

INSTRUCTIONS FOR USE:

- To maximize advantages of the surface, wet the catheter before use.
- Insert the flexible introducer into the catheter while approximately aligning the curved tips of the catheter and introducer.
- Insert the catheter using the over-the-guidewire technique. Tract over-dilation will ease placement.
- When the catheter is positioned, remove the flexible introducer.
- Attach the catheter funnel to drainage bag, suction source or syringe, using standard tapered connectors.

STERILE EO		0086	Cautión: Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician.
Sterile if package is unopened and undamaged. Non-pyrogenic. For single use only. The reuse of this single-use device can lead to patient infection and/or device malfunction.			
URESIL and ULTI-flo are registered trademarks of UreSil, LLC. GP is a trademark of UreSil, LLC.			
TO REORDER: Call 1-800-538-7374 (U.S. only) Fax: 1-847-982-0106			
EMERGO EUROPE Prinsessegracht 20 2514 AP The Hague The Netherlands			
8-19-253 Rev. 08/22/2017			

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