
UreSil®
Aspiration/Flush Adapter
Luer Lock Systems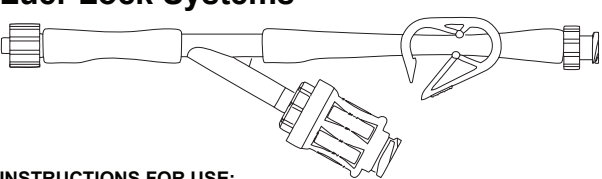

INSTRUCTIONS FOR USE:

1. For catheter maintenance, (aspiration and flushing), attach the Aspiration/Flush (A/F) Adapter to the catheter hub.
2. Connect the bag tube from the selected drainage bag to the luer connector of the A/F Assembly nearest the clamp.
3. Swab the silicone seal of the needleless port in accordance with facility protocol. Attach a male slip luer or male luer lock syringe to the needleless port by grasping the valve and firmly pushing and twisting the syringe onto the port until tight.

CAUTION: Do not use needle to access the needleless port.

4. To flush or aspirate the catheter, clamp the tube with the supplied clamp and complete the aspiration/flush as required. Unclamp the tube when complete and remove the syringe as required by twisting counterclockwise until loose. Do not cap the needleless port. The device is closed when not engaged. For subsequent connections repeat from step three (3).

Sterile if package is unopened and undamaged. Non-pyrogenic. For single use only. The reuse of this single-use device can lead to patient infection and/or device malfunction. Caution: Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician. UreSil is a registered trademark of UreSil, LLC. U.S. Patent no. 8,177,773 TO REORDER: Call 1-800-538-7374 (U.S. only) Fax: 1-847-982-0106

UreSil, LLC,
5418 W. Touhy Ave, Skokie, IL 60077 U.S.A.
EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

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