
UreSil® POSI-STICK® 2 One-Step Introducer Set

en

INTENDED USE:

This device is intended for placement of a percutaneous drainage catheter.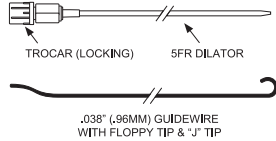

INSTRUCTIONS FOR USE:

1. Insert the trocar with the dilator into the drainage site using standard technique.
2. Remove the trocar leaving the flexible dilator in place.
3. Advance the guidewire through the dilator into the drainage site.
4. Remove the dilator leaving the guidewire in place.

CAUTION:

1. Withdrawal of guidewires should be smooth and without resistance. If resistance is felt, carefully remove wire(s) and system as a unit.
2. Avoid bending, kinking, and uses other than percutaneous catheter placement.
3. Guidewires should be routinely inspected prior to use. Should any deformities be present, discard. Do not use deformed guidewires.
4. Should resistance occur during insertion, DO NOT advance the guidewire.
5. Avoid withdrawing a guidewire through a metal needle. The teflon coating can shear against the needle bevel.

STERILE EO 0086

Sterile if package is unopened and undamaged. Non-Pyrogenic. For single use only. The reuse of this single-use device can lead to patient infection and/or device malfunction. Caution: Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician. UreSil and POSI-STICK are registered trademarks of UreSil, LLC. TO REORDER: Call 1-800-538-7374 (U.S. only) Fax: 1-847-982-0106

UreSil, LLC.
5418 W. Touhy Ave., Skokie, IL 60077 U.S.A.
EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

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