EN Collection Bag	EN Suction Bellows
FR Sac collecteur	FR Soufflet d'aspiration
DE Sammelbeutel	DE Saugbälge
ES Bolsa de recogida	ES Fuelle de succión
TR Toplama Poşeti	TR Emici Körükler
IT Borsa di raccolta	IT Soffiello di aspirazione
NL Oprvangzak	NL Balg
SE Uppsamlingspåse	SE Sugbälge
DA Opsamlingspose	DA Sugebälge
CN 收集袋	CN 吸引波纹管
PT Saco Coletor	PT Foles de Aspiração

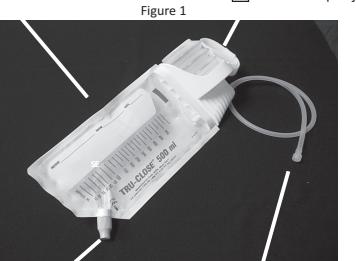
Figure 1

EN Empty Port (if present)	EN Luer Lock
FR Port vide (s'il y a lieu)	FR Raccord luer
DE Leerer Port (falls vorhanden)	DE Luer-Lock-Anschluss
ES Boquilla de vaciado (si dispone de ella)	ES Cierre Luer
TR Boş Port (mevcutsa)	TR Luer Kilidi
IT Aperatura di svuotamento (se presente)	IT Chiusura Luer
NL Afvoerventiel (indien aanwezig)	NL Luer Lock
SE Tömningsöppning (om sådan finns)	SE Luer Lock
DA Tømningssport (hvis til stede)	DA Luerlås
CN 空端口 (如果有)	CN 鲁尔锁
PT Porta Vazia (se presente)	PT Luer Lock

Figure 2

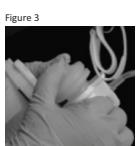
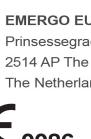
Figure 3

REP

0086

REP

Item Number

Manuf. Lot

Quantity

②

For Single Use Only

STERILE EO

Manufactured By: UreSil, LLC,
5418 W. Touhy Ave., Skokie, IL
60077 U.S.A.Telephone: 1-800-538-7374 (U.S. only)
Fax: 1-847-982-0106Contents sterile unless package has
been opened or damaged.
Sterilized by ethylene oxide.UreSil and TRU-CLOSE are registered
trademarks of UreSil, LLC.

FR Téléphone	Fax	Email	Website
DE Telefon	Fax	Email	Website
ES Teléfono	Fax	E-mail	Página web
TR Telefon	Faks	E-posta	Web sitesi
IT Telefono	Fax	Email	Sito Web
NL Telefoon	Fax	E-mail	Website
SE Telefon	Fax	E-post	Webplats
DA Telefon	Fax	E-mail	Hjemmeside
CN 电话	传真	电子邮件	网址
PT Telefone	Fax	E-mail	Website

EN UreSil® TRU-CLOSE® Suction Drainage System

EN

USA CAUTION:
Federal Law (U.S.A.) restricts this device to sale
by or on the order of a physician.

DEVICE DESCRIPTION:
Suction Drainage System
with Empty Port (if present)

INTENDED USE:

For collecting drainage fluids
This product is intended for short-term use (more
than 24 hours and less than 30 days).

WARNINGS/PRECAUTIONS:
Discard the entire system when the bag is full or
when otherwise indicated.

Sterile if package is unopened and undamaged.

For single use only.

The reuse of this single-use device can lead to
patient infection and/or device malfunction.

Bags without empty ports only: Do not attempt
to empty the bag during use. The bag can be
emptied prior to disposal if desired by cutting off
the bottom corner of the bag.

INSTRUCTIONS FOR USE:

- Close the empty port prior to use (if present).
- Connect the drainage catheter to the luer lock
as shown in Figure 2.

NOTE: For post-surgical drainage, remove the
luer lock connector and attach the
appropriate barbed connector (if
included) for the wound drain being used
(if applicable).

- If altering the bag tube length, remove the
tube from the connector at the top of the
“suction bellows” housing, cut the tubing and
re-attach.
- Suspend the bag at a location below the
drainage site.
- Compress the bellows to aspirate fluid from
the catheter as shown in Figure 3.
- When bellows is full, repeat step 5 to
reactivate suction.

UreSil and TRU-CLOSE are registered
trademarks of UreSil, LLC.

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