
UreSil® Drainage Catheter

en


Catheter	Intended Use
General Purpose (GP/Mini-pigtail®)	Percutaneous drainage of abscess, nephrostomy and other fluids.
NEPHRO-CATH™	Percutaneous drainage of nephrostomy and other fluids.
CHOLE-CATH® Biliary	Percutaneous transhepatic biliary drainage.

See the pouch label on this product for specific contents.

NON-LOCKING CATHETERS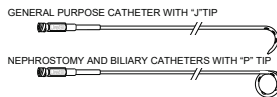

HARDWARE

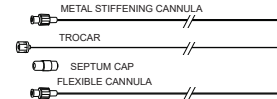INSTRUCTIONS FOR USE:

- To maximize advantages of the surface, WET THE CATHETER before use.
- Straighten the catheter when inserting the cannula (and trocar if appropriate).
- Insert the catheter using the over-the-guidewire technique. Tract over-dilation will ease placement. Catheters with a trocar included can also be inserted using direct puncture.
- For pigtail catheters, rotate the catheter clockwise while removing the cannula to form the pigtail.
- When the catheter is positioned, remove the hardware and fix the catheter in place.
- For biliary and nephrostomy catheters, internal drainage can be achieved by placing the septum cap into the catheter hub. The septum cap contains a self-sealing needle puncture port for flushing.

Sterile if package is unopened and undamaged. Non-pyrogenic. For single use only. The reuse of this single-use device can lead to patient infection and/or device malfunction. Caution: Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician.

URESIL, MINI-PIGTAIL and CHOLE-CATH are registered trademarks of UreSil, LLC. GP, and NEPHRO-CATH are trademarks of UreSil, LLC.

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