UreSil® THORA-VENT® REF: TVST Suction Tubing for Thoracic Vent

INSTRUCTIONS FOR USE:
If attachment to a suction system is desired, remove the suction tubing set from its pouch. Close the clamp on the tubing set while the set is being connected. Insert the cannula through the self-sealing port on the Thoracic Vent and lock it in place by twisting the luer lock. Attach the funnel to the suction system and then open the clamp. Appropriate evacuating pressures should be used. Do not exceed a 14.7 mm Hg (20 cm water) vacuum. Always clamp the suction tubing set when suction is not being applied to the Thoracic Vent. If the suction tubing set is no longer required, remove it and immediately seal the self-sealing port with the tethered cap.

WARNINGS/COMPLICATIONS:
Always clamp the suction tubing set when suction is not being applied to the Thoracic Vent.

STERILE EO 0086

Sterile if package is unopened and undamaged. Non-pyrogenic. For single use only. Caution: Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician. TO REORDER: Call 1-800-538-7374 (U.S.A. only) Fax: (847) 982-0106 UreSil and THORA-VENT are registered trademarks of UreSil, LLC. U.S. patent no. 6,770,063

UreSil, LLC.
5418 W. Touhy Ave, Skokie, IL 60077 U.S.A.
EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

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