

# Evolution™ PRE-VAC WOUND DRAINAGE BOTTLE

1000 mL



Refer to the label




Caution: Federal law (U.S.A.) restricts this device to sales by or on the order of a physician.

Re-use of this product will cause the risk of cross infection and unpredictable health threat.

This product is not made with natural rubber latex.

## Directions for use

1. Open the pouch.
2. Inspect the vacuum indicator is at "MAX" line. if not, do not use this product.
3. CLOSE CONNECTING TUBE CLAMP FIRST! Attach Male Luer Connector of the connecting tube (if provided) to Female Luer of patient drainage device.
4. After checking all connections are secure, open the white slide clamp on the bottle to activate vacuum.
5. Open connecting tube clamp to begin drainage procedure.
6. Repeat steps 1 to 5 with new bottle if vacuum indicator is at "MIN" Line, or bottle is almost full and further drainage is required.

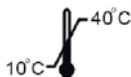

**Manufactured By : PACIFIC HOSPITAL SUPPLY CO., LTD.**

4F., NO. 160, DAYE ROAD, BEITOU DISTRICT, TAIPEI 112, TAIWAN

Tel : 886 2-2895-5050 Fax : 886 2-2897-8282

E-mail : [pahsco@pahsco.com.tw](mailto:pahsco@pahsco.com.tw)

Made In Taiwan

**Distributed by:**

**URESIL, LLC**

5418 W. Touhy Ave.

Skokie, IL 60077

USA



**Imported By:**

**MEDQUEST MEDICAL INC.**

50 Chisholm Drive, Unit 2

Ingersoll, Ontario N5C2C6

CANADA

[www.medquestmedical.com](http://www.medquestmedical.com)



TO REORDER:

Call: 1-800-538-7374

Fax: 1-847-982-0106

[www.uresil.com](http://www.uresil.com)

Evolution is a trademark of UreSil, LLC. UreSil is a registered trademark of UreSil, LLC.