

## FOR BEST RESULTS:


Clean application site with an alcohol wipe and/or non-moisturizing soap prior to application.

### Contents:

- 1 Hydrocolloid Universal Securement Device

### For use on intact skin.

If redness or irritation occurs, stop use and consult a healthcare practitioner.

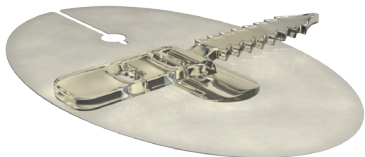

**See reverse for complete application instructions.**



**Manufactured by BioDerm®**, Inc.  
12320 73rd Court N, Largo, FL, 33773 USA

**Distributed by UreSil, LLC**  
5418 W. Touhy Ave, Skokie, IL 60077 USA  
Phone: (800) 538-7374 | [www.UreSil.com](http://www.UreSil.com)


LT 94069 IR

# Applying Fortitude™




## STEP 1:

Make sure application site is clean and dry. If possible, clean entire application area with alcohol wipe and allow to dry completely.

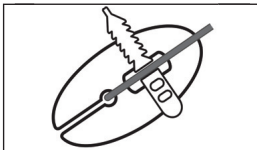

## STEP 2:

Place device on skin with the tube resting at top of the slot in oval base.

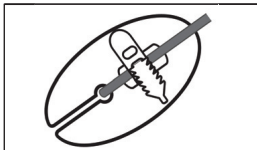

## STEP 3:

Remove release papers and press to skin. Rest the tube on the soft, no-slip grips. Pull tight to secure.

**ATTENTION:** Following application, leave a one inch space around the Fortitude™ device when using any cleansing products, creams or lotions. Contact with the hydrocolloid will result in early dislodgement of the securement device.

**REMOVAL:** Freederm™ is recommended to remove Fortitude. Soak with warm water or wet washcloth until Fortitude is milky white and rolls gently from the skin. Spray the edge of the adhesive, lifting gently from the skin as you go. Continue to spray under the adhesive until completely removed. **Never pull or tear from the skin.**