
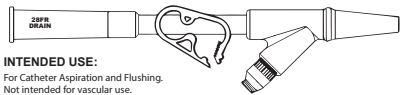
UreSil® Aspiration/Flush Adapter Large Bore Systems

en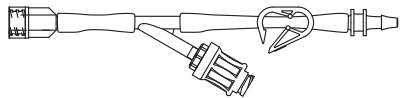

INTENDED USE:

For Catheter Aspiration and Flushing.
Not intended for vascular use.
This product is intended for short-term use (more than 24 hours and less than 30 days).

INSTRUCTIONS FOR USE:

1. For catheter maintenance, (aspiration and flushing), attach the Aspiration/Flush (A/F) Adapter to the Large Bore catheter hub (ex. UreSil's ULTI-flo LBL2 locking or LBNL non-locking).
2. Remove the connector from the selected drainage bag. Reconnect the bag tube to the barbed or funneled connector of the A/F Assembly (next to clamp).
3. Swab the silicone seal of the needleless port in accordance with facility protocol. Attach a male slip luer or male luer lock syringe to the needleless port by grasping the valve and firmly pushing and twisting the syringe onto the valve until tight.
4. To flush or aspirate the catheter, clamp the tube with the supplied clamp and complete the aspiration/flush as required. Unclamp the tube when complete and remove the syringe as required by twisting counter clock-wise until loose. Do not cap the valve. The device is closed when not engaged. For subsequent connections repeat from step three (3).

Sterile if package is unopened and undamaged.
The reuse of this single-use device can lead to patient infection and/or device malfunction. UreSil is a registered trademark of UreSil, LLC. www.uresil.com
U.S. Patent nos. 8,177,773, DES, 585, 134S
TO REORDER: Call 1-800-538-7374 (U.S. only) Fax: 1-847-982-0106

Rx Only

Manufactured by:
UreSil, LLC,
5418 W. Touhy Ave, Skokie, IL 60077 U.S.A.
Authorized Representative in European Community:
EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

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