UreSil® Drainage Catheter

en

Intended Use

Percutaneous drainage of abscess, nephrostomy and other fluids.
This product is intended for short-term use (more than 24 hours and less than 30 days).

See the pouch label on this product for specific contents.


NON-LOCKING CATHETERS

GENERAL PURPOSE CATHETER WITH "J" TIP

HARDWARE

METAL STIFFENING CANNULA

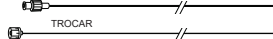TROCAR

FLEXIBLE CANNULA

NOTE: Depth marks (if present) are not intended to measure- for reference only.

INSTRUCTIONS FOR USE:

- To maximize advantages of the surface, WET THE CATHETER before use.
- Straighten the catheter when inserting the cannula (and trocar if appropriate).
- Insert the catheter using the over-the-guidewire technique. Tract over-dilation will ease placement. Catheters with a trocar included can also be inserted using direct puncture.
- For pigtail catheters, rotate the catheter clockwise while removing the cannula to form the pigtail.
- When the catheter is positioned, remove the hardware and fix the catheter in place.

CAUTION:

- When engaging the catheter luer lock, do not overtighten. Overtightening the luer can result in luer crack which may require catheter replacement.
- Dispose of sharps using approved sharps container in accordance with applicable regulations and institutional policy.

Sterile if package is unopened and undamaged. The reuse of this single-use device can lead to patient infection and/or device malfunction.

URESIL, MINI-PIGTAIL and CHOLE-CATH are registered trademarks of UreSil, LLC. GP, and NEPHRO-CATH are trademarks of UreSil, LLC.
TO REORDER: Call 1-800-538-7374 (U.S. only) Fax: 1-847-982-0106
www.uresil.com

Manufactured by: UreSil, LLC,
5418 W. Touhy Ave, Skokie, IL 60077 U.S.A.

Rx Only

Authorized Representative in European Community:
EMERGO EUROPE
Prinsessegracht 20, 2514 AP The Hague
The Netherlands

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© 19-258 Rev. 08/24/2018