
UreSil® POSI-STICK® 2 One-Step

en

Introducer Set INTENDED USE:

This device is intended for placement of a percutaneous drainage catheter.

This product is intended for transient use (less than 24 hours).

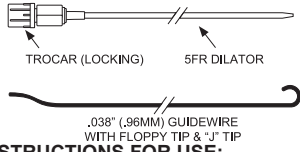INSTRUCTIONS FOR USE:

1. Insert the trocar with the dilator into the drainage site using standard technique.
2. Remove the trocar leaving the flexible dilator in place.
3. Advance the guidewire through the dilator into the drainage site.
4. Remove the dilator leaving the guidewire in place.

CAUTION:

1. Withdrawal of guidewires should be smooth and without resistance. If resistance is felt, carefully remove wire(s) and system as a unit.
2. Avoid bending, kinking, and uses other than percutaneous catheter placement.
3. Guidewires should be routinely inspected prior to use. Should any deformities be present, discard.
4. Do not use deformed guidewires. Should resistance occur during insertion, DO NOT advance the guidewire.
5. Avoid withdrawing a guidewire through a metal needle. The teflon coating can shear against the needle bevel.
6. Dispose of sharps using approved sharps container in accordance with applicable regulations and institutional policy.

Sterile if package is unopened and undamaged. The reuse of this single-use device can lead to patient infection and/or device malfunction. UreSil and POSI-STICK are registered trademarks of UreSil, LLC.

TO REORDER:
Call 1-800-538-7374 (U.S. only) Fax: 1-847-982-0106
www.uresil.com

Manufactured By:
UreSil, LLC,
5418 W. Touhy Ave., Skokie, IL 60077 U.S.A.

Rx Only

EC REP Authorized Representative in European Community:
EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

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