
UreSil® Aspiration/Flush Adapter Double Clamp en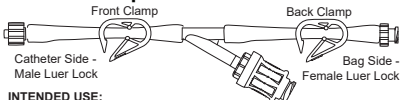

INTENDED USE:
For Catheter Aspiration and Flushing
This product is intended for short-term use
(more than 24 hours and less than 30 days). Not intended for vascular use.

INSTRUCTIONS FOR USE:

- For catheter aspiration and flushing, attach the Dual Flush Adapter to the catheter using the male luer lock.
- Connect the bag to the female luer lock.
- To access the needleless port:
 - Swab the silicone seal of the needleless port in accordance with facility protocol.
 - Attach a male slip or luer lock syringe to the needleless port by grasping the port and firmly pushing and twisting the syringe onto the port until secure.

CAUTION: Do not use needle to access the needleless port.
- To flush or aspirate the catheter, clamp the back clamp and flush and/or aspirate the catheter as appropriate.
- Eliminating syringe contents:
 - The syringe contents can be eliminated by removing the syringe (twist and pull syringe counterclockwise to disconnect from the needleless port), or
 - The syringe contents can be ejected into the bag without disconnecting the syringe. To eject the contents into the bag directly, open the back clamp and close the front clamp. Eject syringe contents into the bag.
- When aspiration and/or flushing is completed, remove the syringe and open both clamps. It is unnecessary to cap the needleless port.

Sterile if package is unopened and undamaged. The reuse of this single-use device can lead to patient infection and/or device malfunction. UreSil is a registered trademark of UreSil, LLC. U.S. Patent no. 6,177,773. www.uresil.com
TO REORDER: Call 1-800-538-7374 (U.S. only) Fax: 1-847-982-0106

Manufactured by:
UreSil, LLC
5418 W. Touhy Ave., Skokie, IL 60077 U.S.A.
Authorized Representative in European Community:
EMERGO EUROPE
Prinssengracht 20
2514 AP The Hague
The Netherlands

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