UreSil® THORA-VENT® EN Suction Tubing for Thoracic Vent

Intended Use:
For use with Thoracic Vent.
This product is intended for short-term use (more than 24 hours and less than 30 days).

INSTRUCTIONS FOR USE:

If attachment to a suction system is desired, remove the suction tubing set from its pouch. Close the clamp on the tubing set while the set is being connected. Insert the cannula through the self-sealing port on the Thoracic Vent and lock it in place by twisting the luer lock. Attach the funnel to the suction system and then open the clamp.

Appropriate evacuating pressures should be used. Do not exceed a 14.7 mm Hg (20 cm water) vacuum. Always clamp the suction tubing set when suction is not being applied to the Thoracic Vent. If the suction tubing set is no longer required, remove it and immediately seal the self-sealing port with the tethered cap.

WARNINGS/COMPLICATIONS:

Always clamp the suction tubing set when suction is not being applied to the Thoracic Vent.

Sterile if package is unopened and undamaged. The reuse of this single-use device can lead to patient infection and/or device malfunction.

Rx Only

TO REORDER:
Call 1-800-538-7374 (U.S.A. only) Fax: (847) 982-0106
www.uresil.com

UreSil and THORA-VENT are registered trademarks of UreSil, LLC. U.S. patent no. 6,770,063

Manufactured by:
UreSil, LLC,
5418 W. Touhy Ave., Skokie, IL 60077 U.S.A.

Authorized Representative in European Community:
EC REP
EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

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