
UreSil® Drainage Catheter

en

Catheter	Intended Use
General-purpose (GP™ Mini-pigtail®)	Percutaneous drainage of abscess, nephrostomy and other fluids.
NEPHRO-CATH™ Nephrostomy	Percutaneous drainage of nephrostomy and other fluids.
CHOLE-CATH® Biliary	Percutaneous transhepatic biliary drainage.

This product is intended for short-term use (more than 24 hours and less than 30 days). **See the pouch label on this product for specific contents.**

LOCKING CATHETERS:
GP, NEPHROSTOMY AND BILIARY CATHETERS WITH LOCKING PIGTAIL

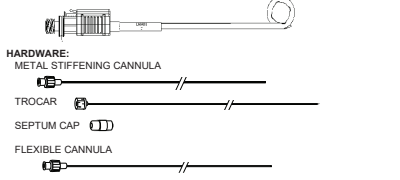Note: Depth marks (if present) are not intended to measure- for reference only.

INSTRUCTIONS FOR USE:

- To maximize advantages of the surface, WET THE CATHETER before use.
- Straighten the catheter when inserting the cannula (and trocar if appropriate). For locking catheters, pull the suture to remove slack before inserting the cannula.
- Insert the catheter using the over-the-guidewire technique. Tract over-dilation will ease placement. Catheters with a trocar included can also be inserted using direct puncture.
- For locking catheters, rotate the catheter clockwise while removing the cannula to form the pigtail.
- When the catheter is properly positioned, remove the hardware.
- For locking pigtail catheters, pull the suture to close the pigtail. Depending upon whether or not suture storage is desired, lock the pigtail as follows:
 - Option A: Locking the pigtail without storing suture: Move the locking slide proximally until an audible "click" is heard. The suture is now locked. Trim away excess suture at the exit point at the proximal end of the hub.
 - Option B: Locking the pigtail with suture storage: Wind approximately 15 cm of suture into the suture channel for later use when removing or repositioning the catheter. To wrap the suture, pull it straight and then hook it in the notch at the bottom of the suture channel. After making the last wind, wrap the suture around the suture retention post. Move the locking slide proximally until an audible "click" is heard. The suture is now locked.
- To remove the catheter, unlock the suture retention mechanism. To unlock the mechanism, depress the tab using an instrument and move the slide distally. If no suture was stored, the suture will now be released. If suture was stored, unwind the suture to complete the release. Cut the suture at the exit point at the proximal end of the hub and pull the catheter out to release the pigtail.
- For biliary and nephrostomy catheters, internal drainage can be achieved by placing the septum cap on the catheter hub. The septum cap contains a self-sealing needle puncture port for flushing.

CAUTION:

- When engaging the catheter luer lock, do not overtighten. Overtightening the luer can result in luer crack which may require catheter replacement.
- Dispose of sharps using approved sharps container in accordance with applicable regulations and institutional policy.

STERILE EO
For single use only
CE 2797
Rx Only

Sterilized using ethylene oxide

Sterile if package is unopened and undamaged. The reuse of this single-use device can lead to patient infection and/or device malfunction.

URESIL, MINI-PGTAİL and CHOLE-CATH are registered trademarks of URESIL, LLC. GP and NEPHRO-CATH are trademarks of URESIL, LLC. U.S. patent no. 8,177,877 and 8,073,068/1.

TO REORDER: Call 1-800-538-7374 (U.S. only) Fax: 1-847-982-0106 www.uresil.com

Manufactured by: UreSil, LLC
5418 W. Touhy Ave., Skokie, IL 60077 U.S.A.

Authorized Representative in the European Community:
EMERGO EUROPE
Pflanzengraben 20, 2514 AP The Hague
The Netherlands

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