UreSil® ULTI-flo® en


General-Purpose Drainage Catheter System with Aspiration/Flush Assembly

Catheter	Intended Use
General-purpose (ULTI-flo)	Percutaneous drainage of abscess, nephrostomy, and other fluids.

This product is intended for short-term use (more than 24 hours and less than 30 days).

See the pouch label on this product for specific contents

LOCKING CATHETERS:
ULTI-flo General-Purpose Drainage Catheter System with Aspiration/Flush Assembly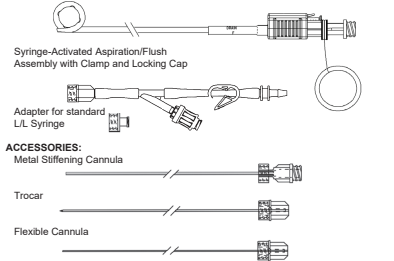

NOTE: Depth marks (if present) are not intended to measure- for reference only.

INSTRUCTIONS FOR USE:


1. To maximize advantages of the surface, WET THE CATHETER before use.
2. Straighten the catheter when inserting the cannula (and trocar if appropriate). For locking catheters, pull the suture to remove slack before inserting the cannula.
3. Insert the catheter using the over-the-guidewire technique. Tract over-dilation will ease placement. Catheters with a trocar included can also be inserted using direct puncture.
4. For locking catheters, rotate the catheter clockwise while removing the cannula from the pigtail.
5. When the catheter is properly positioned, remove the hardware.
6. For locking pigtail catheters, pull the suture to close the pigtail. Depending upon whether or not suture storage is desired, lock the pigtail as follows:

Option A: Locking the pigtail without storing suture: Move the locking slide proximally until an audible "click" is heard. The suture is now locked. Trim away excess suture at the exit point at the proximal end of the hub.

Option B: Locking the pigtail with suture storage: Wind approximately 15 cm of suture into the suture channel for later use when removing or repositioning the catheter. To wrap the suture, pull it straight and then hook it in the notch at the bottom of the suture channel. After making the last wind, wrap the suture around the suture retention post. Move the locking slide proximally until an audible "click" is heard. The suture is now locked.


7. To remove the catheter, unlock the suture retention mechanism. To unlock the mechanism, depress the tab using an instrument and move the slide distally. If no suture was stored, the suture will not be released. If suture was stored, unwind the suture to complete the release. Cut the suture at the exit point at the proximal end of the hub and pull the catheter out of the patient. In order to maintain control of the suture, do not cut the catheter to release the pigtail.
8. To aspirate the catheter after insertion attach the luer-lock (L/L) syringe adapter to the hub.
9. For catheter maintenance (aspiration and flushing), attach the Aspiration/Flush (A/F) assembly to the large bore catheter hub.
10. Remove the standard adapter from the selected drainage bag. Reconnect the bag tube to the slip connector of the A/F assembly.
11. Swab the silicone seal of the needletest port in accordance with facility protocol. Attach a male slip luer or male luer lock syringe to the needletest port by grasping the valve and firmly pushing and twisting the syringe onto the valve until tight.
12. To flush or aspirate the catheter, clamp the tube with the supplied clamp and complete the aspiration/flush as required. Uncrimp the tube when complete and remove the syringe as required by twisting counter-clockwise until loose. Do not cap the valve. The device is closed when not engaged. For subsequent connections repeat from step 11.

CAUTION:
1. Do not use needing to access the valve port.
2. When engaging the catheter luer lock, do not overtighten. Overtightening the luer can result in luer crack which may require catheter replacement.
3. Dispose of sharps using approved sharps container in accordance with applicable regulations and institutional policy.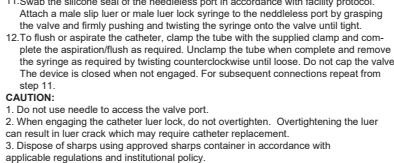

Sterile if package is unopened and undamaged. The reuse of this single-use device can lead to patient infection and/or device malfunction. URESIL, ULTI-flo are registered trademarks of URESIL, LLC. GP is a trademark of URESIL, LLC.
U.S. patent no's. 8,177,773; 9,079,006B1; DES, 585,134S
TO REORDER: Call 1-800-538-7374 (U.S. only)

Fax: 1-847-982-0106 www.uresil.com
Manufactured by: Uresil, LLC
118 W. Touhy Ave., Skokie, IL 60077 U.S.A.
Authorized Representative in European Community:
EMERGO EUROPE
Prinsessegracht 20, 2514 AP The Hague
the Netherlands

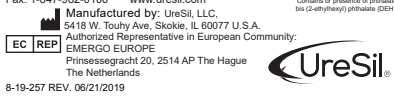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