
EN Collection Bag  
FR Sac collecteur  
DE Sammelbeutel  
ES Bolsa de recogida  
TR Toplama Poşeti  
IT Borsa di raccolta  
NL Opvangzak  
SE Uppsamlingspåse  
DA Opsamlingspose  
CN 收集袋  
PT Saco Coletor

EN Suction Bellows  
FR Soufflet d'aspiration  
DE Saugbälge  
ES Fuelle de succión  
FR Emcici Kőrükler  
IT Soffietto di aspirazione  
NL Balg  
SE Sugbälg  
DA Sugebælg  
CN 吸引波纹管  
PT Foles de Aspiração

Figure 1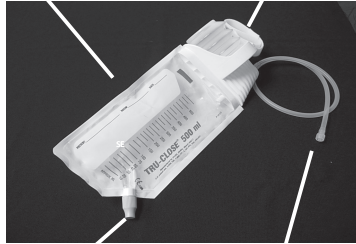

EN Empty Port (if present)  
FR Port vide (s'il y a lieu)  
DE Leerer Port (falls vorhanden)  
ES Boquilla de vaciado (si dispone de ella)  
TR Boş Port (mevcutsa)  
IT Apertura di svuotamento (se presente)  
NL Afvoerventiel (indien aanwezig)  
SE Tömningsspning (om sådan finns)  
DA Tømningsport (hvis til stede)  
CN 空端口 (如果有)  
PT Porta Vazia (se presente)

EN Luer Lock  
FR Raccord luer  
DE Luer-Lock-Anschluss  
ES Cierre Luer  
TR Luer Kiliti  
IT Chiusura Luer  
NL Luer Lock  
SE Luer Lock  
DA Luerlås  
CN 鲁尔锁  
PT Luer Lock

Figure 2

Figure 3

STERILE EO  2797 Rx Only  
Sterilized using ethylene oxide For single use only

Manufactured By: UreSil, LLC  
5418 W. Touhy Ave., Skokie, IL  
60077 U.S.A.

EC REP Authorized Representative in European Community:  
EMERGO EUROPE  
Prinsessegracht 20, 2514 AP The Hague  
The Netherlands

To Reorder: Call 1-800-538-7374 (U.S.A. only)  
Fax: (847) 982-0106  
www.uresil.com

8-19-273 rev. 06/11/2019

## UreSil® TRU-CLOSE® Suction Drainage System

**DEVICE DESCRIPTION:**  
Suction Drainage System  
with Empty Port (if present)

**INTENDED USE:**  
For collecting drainage fluids  
This product is intended for short-term use (more than 24 hours and less than 30 days).

**NOTE:** Graduation marks on bag not intended to measure – for reference only

### WARNINGS/PRECAUTIONS:

- Discard the entire system when the bag is full or when otherwise indicated.
- Use care if emptying bag to avoid contact with drained fluids.

**Bags without empty ports only:** Do not attempt to empty the bag during use. The bag can be emptied prior to disposal if desired by cutting off the bottom corner of the bag.

### INSTRUCTIONS FOR USE:

1. Close the empty port prior to use (if present).
  2. Connect the drainage catheter to the luer lock as shown in Figure 2.
- NOTE: For post-surgical drainage, remove the luer lock connector and attach the appropriate barbed connector (if included) for the wound drain being used (if applicable).
3. If altering the bag tube length, remove the tube from the connector at the top of the "suction bellows" housing, cut the tubing and re-attach.
  4. Suspend the bag at a location below the drainage site.
  5. Compress the bellows to aspirate fluid from the catheter as shown in Figure 3.
  6. When bellows is full, repeat step 5 to reactivate suction.

Sterile if package is unopened and undamaged. The reuse of this single-use device can lead to patient infection and/or device malfunction.

UreSil and TRU-CLOSE are registered trademarks of UreSil, LLC.

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