STERILE EO For single use only
 Sterilized using ethylene oxide

CE 2797 Rx Only

Manufactured By: UreSil, LLC
 5418 W. Touhy Ave., Skokie, IL
 60077 U.S.A.

EC REP Authorized Representative
 in European Community:
EMERGO EUROPE
 Prinsessegracht 20, 2514 AP
 The Hague, The Netherlands

To Reorder: Call 1-800-538-7374 (U.S.A. only)
 Fax: (847) 982-0106
 www.uresil.com


UreSil®

8-19-200 rev. 12/09/2019

**UreSil® Optimus®
 600 ml Gravity Drainage Bag**

DEVICE DESCRIPTION:
 600 ml Gravity Drainage Bag

INTENDED USE:
 For collecting drainage fluids.
 This product is intended for short-term use (more than 24 hours and less than 30 days).

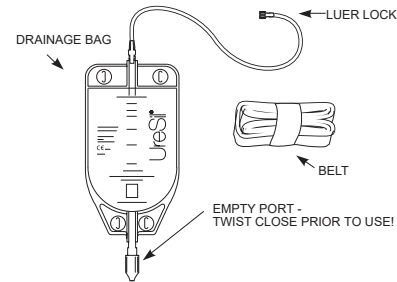NOTE: Graduation marks on bag not intended to measure – for reference only

CAUTION:
 Use care if emptying bag to avoid contact with drained fluids.

INSTRUCTIONS FOR USE:

1. Close the port prior to use.
2. Thread the belt as shown.
3. Connect the drainage catheter to the bag tubing as shown.
4. If altering the bag tube length, using sterile technique, remove the luer lock, cut the tubing and re-attach the luer lock.
5. Suspend the bag at a location below the drainage site as shown.
6. Replace bag every 5 to 7 days or sooner if clinically indicated, for example, if malodorous or damaged.

Sterile if package is unopened and undamaged. The reuse of this single-use device can lead to patient infection and/or device malfunction. This product is intended only for use by trained clinicians.

Optimus and UreSil are registered trademarks of UreSil, LLC.

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