## UreSil® TRU-FIX™ Catheter Fixation Kit

### Intended use:

For externally securing percutaneous catheters. Do not use on patients with known tape or adhesive allergies. This product is intended for short-term use (More than 24 hours and less than 30 days). Not intended for vascular use.

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
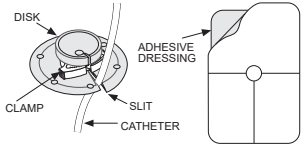
tr

### INSTRUCTIONS FOR USE:


1. Clean and dry the catheter exit site.
2. Position the disk onto the catheter via the slit.
3. Slide the disk onto the skin.
4. Close the clamp completely.
5. Apply the adhesive dressing after removing the release paper.
6. The adhesive patch is expected to adhere for at least 7 days.

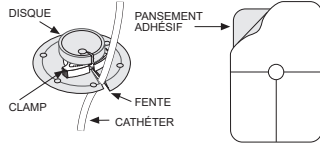If replacement of patch is warranted or desired, remove carefully by peeling away from skin. Clean and dry the catheter exit site and place a new patch (REF TFD-R).

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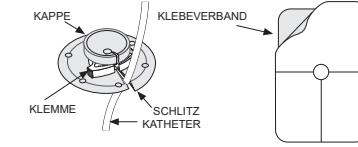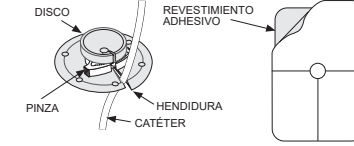
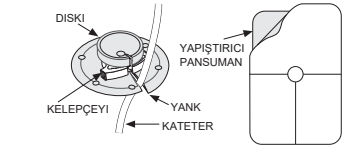
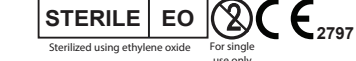
Sterilized using ethylene oxide For single use only  
Sterile if package is unopened and undamaged. The reuse of this single-use device can lead to patient infection and/or device malfunction. This product is intended only for use by trained clinicians. TRU-FIX is a trademark of UreSil, LLC. UreSil is a registered trademark of UreSil, LLC.  
U.S. Patent no. 5,833,666 TO REORDER: Call 1-800-538-7374 (U.S. only) Fax: 1-847-982-0106  
www.uresil.com

Rx Only

Manufactured by: UreSil, LLC,  
5418 W. Touhy Ave, Skokie, IL 60077 U.S.A.

Authorized Representative in European Community:  
EMERGO EUROPE  
Prinsessegracht 20, 2514 AP The Hague  
The Netherlands

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