UreSil® THORA-VENT®

EN

Occlusion Plug

REF: TVOCP

Intended Use:

For use with Thoracic Vent. This product is intended for short-term use (more than 24 hours and less than 30 days).

This package contains instructions for the specific use of the occlusion plug. For additional instructions related to the use of the Thoracic Vent, see the instructions for use that are included in the Thoracic Vent Procedure Trays.

INSTRUCTIONS FOR USE:

Check for continuous air leak: The Thoracic Vent occlusion plug can be used to check for a continuous air leak from the patient's lung. To occlude the Thoracic Vent, insert and secure the occlusion plug to the luer-lock on the self-sealing port.

WARNINGS/COMPLICATIONS:

- Remove the occlusion plug when the check is complete. The Thoracic Vent will not vent air when the occlusion plug is in place. Seal the self-sealing port with the tethered cap after the occlusion plug is removed.
- If the occlusion plug was used to perform the air leak check, remove the occlusion plug when the check is complete. The Thoracic Vent will not vent air when the occlusion plug is in place.

Sterile if package is unopened and undamaged. The reuse of this single-use device can lead to patient infection and/or device malfunction. This product is intended only for use by trained clinicians.
TO REORDER: Call 1-800-538-7374 (U.S.A. only)
Fax: (847) 982-0106 www.uresil.com
UreSil and THORA-VENT are registered trademarks of UreSil, LLC. U.S. patent no. 6,770,063

Manufactured by:
 UreSil, LLC,
5418 W. Touhy Ave, Skokie, IL 60077 U.S.A.
Authorized Representative in European Community:
 EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

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