
UreSil® ULTI-flo® en

General-Purpose Non-Locking Catheter

Catheter	Intended Use
General-purpose (GP™ Non-Locking)	Abscess and other fluid drainage.

This product is intended for short-term use (more than 24 hours and less than 30 days). **See the pouch label on this product for specific contents.**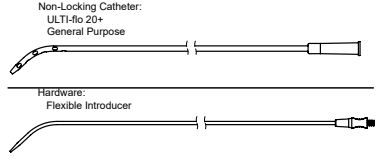

INSTRUCTIONS FOR USE:


- To maximize advantages of the surface, wet the catheter before use.
- Insert the flexible introducer into the catheter while approximately aligning the curved tips of the catheter and introducer.
- Insert the catheter using the over-the-guidewire technique. Tract over-dilatation will ease placement.
- When the catheter is positioned, remove the flexible introducer.
- Attach the catheter funnel to drainage bag, suction source or syringe, using standard tapered connectors.

Sterile if package is unopened and undamaged. The reuse of this single-use device can lead to patient infection and/or device malfunction. This product is intended only for use by trained clinicians.

URESIL and ULTI-flo are registered trademarks of UreSil, LLC. GP is a trademark of UreSil, LLC. TO REORDER: Call 1-800-538-7374 (U.S. only) Fax: 1-847-982-0108 www.uresil.com

Manufactured by: UreSil, LLC, 5418 W. Touhy Ave, Skokie, IL 60077 U.S.A. Rx Only

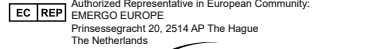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