UreSil® Origin® Nephro-Stent™  
Nephroureteral Stent Catheter

EN

See the pouch label on this product for specific contents.


#### INTENDED USE:

For percutaneous internal/external nephroureteral drainage. This product is intended for short-term use (more than 24 hours and less than 30 days).

#### LOCKING CATHETER: NEPHROURETERAL STENT CATHETER WITH LOCKING PIGTAIL AND RO MARKER

#### HARDWARE: METAL STIFFENING CANNULA

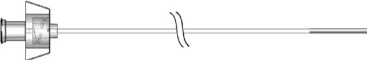#### SEPTUM CAP

#### FLEXIBLE CANNULA

#### OBTURATOR

#### INSTRUCTIONS FOR USE:

- Select and prepare the insertion site using standard technique. Using standard technique, insert a 0.038" diameter guidewire percutaneously through the renal pelvis, ureter, and into the bladder.
- Determine the appropriate Nephroureteral Stent Catheter pigtail-to-pigtail length using one of two techniques:
  - Measure the ureter on x-ray and select the appropriate Nephroureteral Stent Catheter pigtail-to-pigtail length. The stent catheter should be sufficient to allow the proximal pigtail to form in the renal pelvis, while the distal pigtail forms in the bladder.
  - Insert an angiographic catheter or other appropriate device over the guidewire and into the bladder. Mark the device at the skin site. Slowly withdraw the device until the end is located in the renal pelvis. Again, mark the device at the skin site. The distance between the two marks plus an additional 2 cm is the approximate pigtail-to-pigtail length using the same procedure.
- To ease introduction of the Nephroureteral Stent Catheter, the tract may be dilated.
- Remove slack in the retention suture by pulling it out gently until slight resistance is felt.
- Straighten the pigtails prior to advancing the stiffening cannula into the stent catheter. If using the flexible cannula for insertion, first insert the supplied obturator into the flexible cannula. Then, gently advance the reinforced flexible cannula into the stent catheter.
- Lock the stiffening cannula into the stent catheter hub. If the flexible cannula was used, remove the obturator.
- Advance the stent catheter with the stiffening cannula over the guidewire into the superficial part of the renal pelvis. At the point of entry into the renal pelvis, the cannula is unlocked and held stationary while the stent catheter is advanced further until the distal tip is in the bladder.
- Remove the stiffening cannula.
- Slowly remove the guidewire while rotating the stent catheter clockwise to form the pigtails.
- Pull the stent catheter with the stiffening cannula over the guidewire until the superficial part of the renal pelvis is reached. The stent catheter is now locked. Option A: Locking the pigtail without storing suture: Move the locking slide proximally until an audible "click" is heard. The suture is now locked. Trim away excess suture at the exit point at the proximal end of the hub. Option B: Locking the pigtail with suture storage: Wind approximately 15 cm of suture into the suture channel for later use when removing or repositioning the catheter. To wrap the suture, pull it straight and then hook it in the notch at the bottom of the suture channel. After making the last wind, wrap the suture around the suture retention post. Move the locking slide proximally until an audible "click" is heard. The suture is now locked.
- To remove the catheter, unlock the suture retention mechanism. To unlock the mechanism, depress the tab using an instrument and move the slide distally. If no suture was stored, the suture will now be released. If suture was stored, unwind the suture to complete the release. Cut the suture at the exit point at the proximal end of the hub and pull the catheter out of the patient. In order to maintain control of the suture, do not cut the catheter to release the pigtail.
- Internal drainage can be achieved by placing the septum cap on the stent catheter hub. The septum cap contains a self-sealing needle puncture port for flushing.

#### CAUTION:

- The stent catheter may rapidly obstruct where chronic stone formers more rapidly cause encrustations.
- When engaging the catheter luer lock, do not overtighten. Do not use instruments to tighten connections. Overtightening the luer can result in luer crack which may require catheter replacement.
- Dispose of sharps using approved sharps container in accordance with applicable regulations and institutional policy.

Sterile if package is unopened and undamaged. The reuse of this single-use device can lead to patient infection and/or device malfunction. This product is intended only for use by trained clinicians. URESIL and Origin are registered trademarks of UreSil, LLC. NEPHRO-STENT is a trademark of UreSil, LLC. U.S. patent no's. 8,177,773; 9,079,006B1.

**STERILE EO**   
Sterilized using ethylene oxide For single use only Non-pyrogenic

www.uresil.com  
TO REORDER: Call 1-800-538-7374 (U.S. only) Fax: 1-847-982-0106  
Manufactured by: UreSil, LLC,  
5418 W. Touhy Ave, Skokie, IL 60077 U.S.A. **Rx Only**

**EC REP** Authorized Representative in European Community:  
EMERGO EUROPE  
Prinsessegracht 20,  
2514 AP The Hague  
The Netherlands 

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