
EN Collection Bag
FR Sac collecteur
DE Sammelbeutel
ES Bolsa de recogida
TR Toplama Poşeti
IT Borsa di raccolta
NL Opvangzak
SE Uppsamlingspåse
DA Opsamlingspose
CN 收集袋
PT Saco Coletor

EN Suction Bellows
FR Soufflet d'aspiration
DE Saugbälge
ES Fuelle de succión
TR Emici Kөрükler
IT Soffietto di aspirazione
NL Balg
SE Sugbälg
DA Sugebælg
CN 吸引波纹管
PT Foles de Aspiração

Figure 1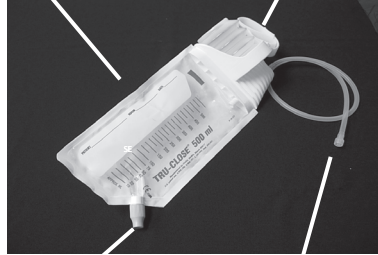

EN Empty Port (if present)
FR Port vide (s'il y a lieu)
DE Leerer Port (falls vorhanden)
ES Boquilla de vaciado (si dispone de ella)
TR Boş Port (mevcutsa)
IT Apertura di svuotamento (se presente)
NL Afvoerventiel (indien aanwezig)
SE Tömningsöppning (om sådan finns)
DA Tømningsport (hvis til stede)
CN 空端口 (如果有)
PT Porta Vazia (se presente)

EN Luer Lock
FR Raccord Luer
DE Luer-Lock-Anschluss
ES Cierre Luer
TR Luer Kilidi
IT Chiusura Luer
NL Luer Lock
SE Luer Lock
DA Luerlås
CN 鲁尔锁
PT Luer Lock

Figure 3

STERILE EO For single use only
2797 Rx Only

Manufactured By: UreSil, LLC
5418 W. Touhy Ave., Skokie, IL
60077 U.S.A.

Authorized Representative in European Community:
EMERGO EUROPE
Prinsessegracht 20, 2514 AP The Hague
The Netherlands

To Reorder: Call 1-800-538-7374 (U.S.A. only)
Fax: (847) 982-0106
www.uresil.com

UreSil®

8-19-273 rev. 12/09/2019

UreSil® TRU-CLOSE® Suction Drainage System

DEVICE DESCRIPTION:
Suction Drainage System
with Empty Port (if present)

INTENDED USE:

For collecting drainage fluids

This product is intended for short-term use (more than 24 hours and less than 30 days).

NOTE: Graduation marks on bag not intended to measure – for reference only

WARNINGS/PRECAUTIONS:

- Discard the entire system when the bag is full or when otherwise indicated.
- Use care if emptying bag to avoid contact with drained fluids.

Bags without empty ports only: Do not attempt to empty the bag during use. The bag can be emptied prior to disposal if desired by cutting off the bottom corner of the bag.

INSTRUCTIONS FOR USE:

- Close the empty port prior to use (if present).
- Connect the drainage catheter to the luer lock as shown in Figure 2.

NOTE: For post-surgical drainage, remove the luer lock connector and attach the appropriate barbed connector (if included) for the wound drain being used (if applicable).

- If altering the bag tube length, remove the tube from the connector at the top of the “suction bellows” housing, cut the tubing and re-attach.
- Suspend the bag at a location below the drainage site.
- Compress the bellows to aspirate fluid from the catheter as shown in Figure 3.
- When bellows is full, repeat step 5 to reactivate suction.

Sterile if package is unopened and undamaged. The reuse of this single-use device can lead to patient infection and/or device malfunction. This product is intended only for use by trained clinicians.

UreSil and TRU-CLOSE are registered trademarks of UreSil, LLC.

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