
## UreSil® Aspiration/Flush Adapter

EN


### INTENDED USE:

For Catheter Aspiration and Flushing. Not intended for vascular use. This product is intended for short-term use (more than 24 hours and less than 30 days).

### AFA-LL and DFA-LL for luer lock catheters:

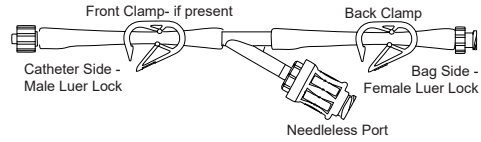### AFA-LB for large bore catheters:
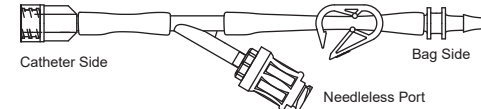
### INSTRUCTIONS FOR USE:

- For catheter maintenance (aspiration and flushing), attach the appropriate adapter to the catheter. The male threaded connector attaches to the catheter.
- Connect the other end of the adapter to a collection bag. For the AFA-LB, remove the luer connector from the bag tubing and attach the bag tubing to the barb.
- Swab the silicone seal of the needleless port in accordance with facility protocol. Attach a male slip luer or male luer lock syringe to the needleless port by grasping the valve and firmly pushing and twisting the syringe onto the valve until tight.
- CAUTION:** Do not use needle to access the needleless port. For all adapters other than the DFA-LL, flush or aspirate the catheter by clamping the tube with the supplied clamp and then aspirating/flushing as required. Uncamp the tube when complete and remove the syringe as required by twisting counter clock-wise until loose. Do not cap the valve. The device is closed when not engaged.
- The DFA-LL is provided with two clamps. When aspirating and flushing with the DFA-LL, close the back clamp and leave the front clamp open. For transferring fluid from the syringe to the bag, open the back clamp and close the front clamp. When aspiration and/or flushing is completed, remove the syringe and open both clamps. It is unnecessary to cap the needleless port.

Sterile if package is unopened and undamaged. The reuse of this single-use device can lead to patient infection and/or device malfunction. This product is intended only for use by trained clinicians. URESIL and ULTI-flo are registered trademarks of UreSil, LLC. U.S. patent no's. 8,177,773; DES. 585,134S

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